TrollByte  Kimera
Endo

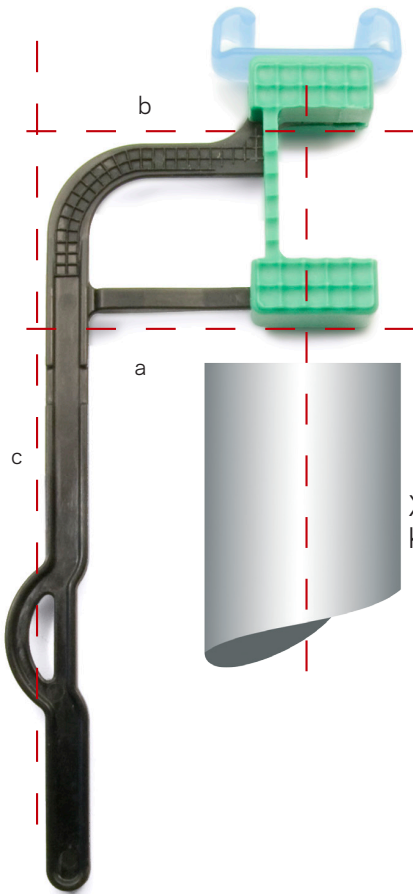
TrollDental

Box 900 • 461 29 Trollhättan • Tel. +46 520 48 90 50 • www.trolldental.com

Basic alignment features

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The tip of the small aiming pin (a) indicates the center of the sensor.

The arm (b) indicates a 90 degree angle to the sensor.

So, the combination of the arm and the small aiming pin makes it easy to determine both vertical and horizontal position of the sensor. This applies when the sensor is centered, as in bitewing images. When capturing anterior and posterior PA's, it is advised to use an aiming ring.

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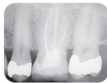
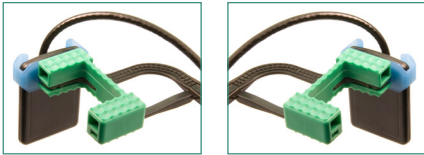
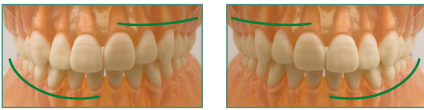
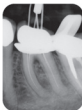
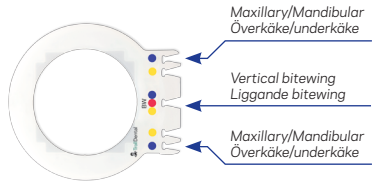
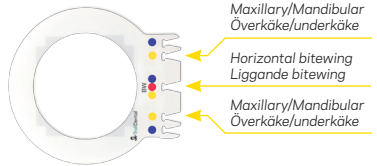
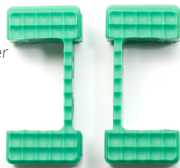
Endo block

The endo blocks are mirrored, one for upper left/lower right jaw and the other for upper right/lower left jaw.

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EN

TrollByte Kimera shall only be used as a holder for dental x-ray. The manufacturer and its dealers are not liable for any other use of the product.

TrollByte Kimera is available in several sizes to fit different sensors, image plate or film. The user is responsible for the selection of size and model.

Always use a sensor cover to protect the sensor while taking x-rays.

Clean prior to first use.

Procedures to clean the product:

- Spirits
- Dishwashing machine/disinfectant
- Autoclave

Autoclave procedure

All included parts are autoclavable and shall be autoclaved separately.

Pre-programmed sterilization cycles according to the manufacturer of the autoclave. Max temperature 134°C (273°F). Place in the middle tray of the autoclave, away from autoclave walls and heating element.

Caution: If pressure or weight from other objects or instruments is applied during autoclaving, the holder may be deformed. Minor deformations can easily be corrected by gently reshaping the holder.

Please note: Any method of sterilization will shorten the life span of plastic parts.

Warranty

TrollByte Kimera is manufactured from high grade polymers. Plastic parts have a limited life span and should be replaced periodically.

Warranty Period for all TrollByte Kimera sensor holders, under normal use, is limited to 12 months from delivery, or 75 autoclave cycles, whichever occurs first.

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